**LINK<sup>®</sup>**

**VARIO-KOPF  
ENDOPROTHESESYSTEM**

***VARIO-CUP  
PROSTHESIS SYSTEM***

**Implantate  
*Implants***

## Systembeschreibung

## Description of the System

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*The LINK® Vario-Cup Prosthesis System consists of UHMWPE acetabular components encased in ultra-smooth polished outer metal shells, for articulation in the bony acetabulum. The implants are to be used in conjunction with femur components of LINK® Total Hip Systems.*

*The LINK® Vario-Cup Prostheses are available in outer diameters ranging from 39 to 65 mm in 1 mm increments. Too small or too large acetabular prostheses proved to result in bone reactions caused due to misloading in the bony acetabulum. By choosing the proper fitting component from the range of finely graduated prosthesis sizes, this complication can be avoided.*

*The LINK® Vario-Cup prostheses are available in inner diameters 32.1, 28.1 and 24.1 mm to be combined with LINK® Hip Prostheses in head sizes 32 mm (acetabular components in outer diameters 47-65 mm), 28 mm (acetabular components in outer diameters 43-65 mm) respectively 24 mm (acetabular components in outer diameters 39-43 mm). Due to manufacturing tolerances which have been matched within the LINK® Hip Systems to allow combination of their components, it is not advisable to use a femur prosthesis manufactured by other sources.*

*For optimal security against luxation between the Vario-Cup prosthesis and the femoral components, an anti-luxation system has been developed for the acetabular components with inner diameters of 28.1 and 32.1 mm. A safety ring of UHMWPE is placed into a slot in the mouth of the polyethylene liner after assembly of acetabular and femoral components. This safety measure can be implemented either before or after implantation of the femoral component, but should be carried out conveniently prior to reposition of the Vario-Cup in the bony acetabulum.*

### Advantages of the System:

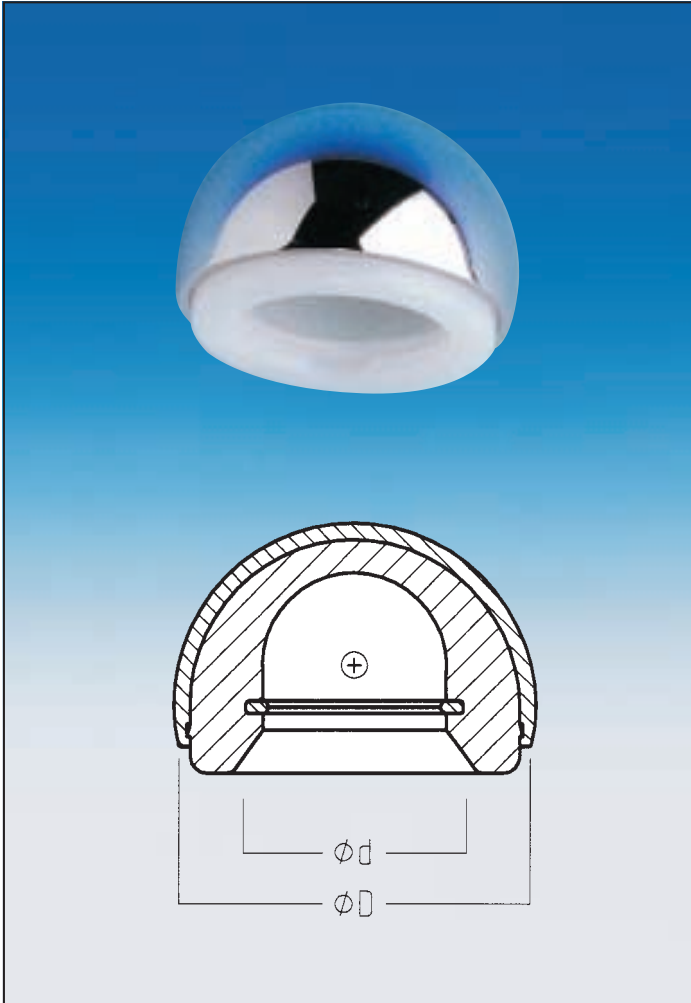
- *The highly polished metal shell eliminates critical reaction of wear between polyethylene and bone.*
- *Less wear in the bony acetabulum as the head of the femoral component articulates primarily within the Vario-Cup.*
- *To accommodate individual patient anatomy a full range of sizes is available for combination with LINK® Standard Hip Prostheses.*
- *Scope for conversion to Total Hip Prosthesis without replacement of the femoral component.*
- *The anti-luxation system provides optimal protection against the risk of dislocation.*
- *Simple disassembly after extraction of the safety ring in case of implant removal.*

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*Endoprosthetic care of medial femoral head fractures in older patients. Especially recommended in cases in which conventional endoprostheses (Moore, Thompson) are indicated. Care of patients who cannot be exposed to the greater trauma of total hip replacement.*

**LINK® Vario-Kopf Pfannen  
– selbstzentrierend –**
**LINK® Vario-Cup Prosthesis  
– self-centering –**

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CoCrMo Alloy to ISO 5832-4 and  
ASTM F 75 and UHMWPE to ISO 5834-2 and  
ASTM F 648

Innen-Ø / Inner Dia. (d):			Außen-Ø Outer Dia. (D) mm
24,1 mm	28,1 mm	32,1 mm	
Art.-Nr. Item-No.	Art.-Nr. Item-No.	Art.-Nr. Item-No.	
107-210/39*#	-	-	39
107-210/40*#	-	-	40
107-210/41*	-	-	41
107-210/42*	-	-	42
107-210/43*	107-060#	-	43
-	107-220/44	-	44
-	107-220/45	-	45
-	107-220/46	-	46
-	107-220/47	107-170#	47
-	107-220/48	107-172#	48
-	107-220/49	107-230/49	49
-	107-220/50	107-230/50	50
-	107-220/51	107-230/51	51
-	107-220/52	107-230/52	52
-	107-220/53	107-230/53	53
-	107-220/54	107-230/54	54
-	107-220/55	107-230/55	55
-	107-220/56	107-230/56	56
-	107-220/57	107-230/57	57
-	107-220/58	107-230/58	58
-	107-220/59	107-230/59	59
-	107-220/60	107-230/60	60
-	107-220/61	107-230/61	61
-	107-220/62	107-230/62	62
-	107-220/63	107-230/63	63
-	107-220/64	107-230/64	64
-	107-220/65	107-230/65	65

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X-ray Template for self-centering  
LINK® Vario-Cup Prosthesis, Set of 4 sheets  
110 % of actual size (not illustrated)

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**The Vario-Cup Prostheses**  
can be combined with different LINK®  
Hip Joint Prosthesis Systems:

**LINK® Lubinus Classic Plus® Hüftprothesensystem**  
**LINK® Lubinus Classic Plus® Total Hip System**

Implantate / Instrumente  
Katalog: 666 dt  
Implants / Instruments  
Catalog: 666 en

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*The Vario-Cup Prosthesis is fitted on to the head of the femoral component.*

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*When inserted into the front opening on the plastic inlay of the prosthesis, the flexible safety ring slides into the slot.*

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*With a little readjustment, the complete circular safety ring is fitted into the slot.*

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*The safety ring to be placed in a slot of the component reduces the entry circumference. Thus prevents the head from luxation.*

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*The safety ring can be removed easily with the help of an angled hook. If the holes in the ends of the ring are not visible, the hook must be used to rotate the ring in the socket until the holes appear. The ring is then engaged on the hook and can now be easily extracted.*

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**CE** The medical products to which we assign the **CE**-mark comply with the council directive 93/42/EEC corresponding to their classification with the respective conformity valuation process.

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All instruments, unless otherwise noted, are made of stainless steel.